ENGLISH

Topex® Prep & Polish™ Paste

Prep & Polish™ Paste is a pumice based cleaning and polishing paste that is free of color, dyes, oils, fragrance and flavors. The product can be used to clean dentin and enamel surfaces prior to adhesive bonding. Prep & Polish™ Paste cleans without negatively impacting bond strength. Additionally, it can be used for routine prophylaxis with patients who prefer or require no additives.

INDICATIONS FOR USE

- 1) Cleaning dentin and enamel surfaces prior to adhesive bonding, etching or bleaching procedures
- 2) Polishing amalgam or composite restorations
- 3) Removing heavy stains from tooth surface
- 4) Routine prophylaxis and polishing of enamel surfaces

CONTRAINDICATIONS

This product should not be used in individuals with a known sensitivity to any component of this product.

WARNINGS

Do not intentionally misuse by swallowing. For professional dental use only.

PRECAUTIONS

Use only as directed. Not for unsupervised home use. Keep out of reach of children.

Do not re-use individual cups or disposable prophylaxis angles on multiple patients.

ADVERSE REACTIONS

No known adverse reactions.

DOSAGE AND ADMINISTRATION

Each unit dose cup contains enough Topex® Prep & Polish™ Paste for a single procedure

(2g/cup). Using a disposable prophylaxis angle, such as the Upgrade® Disposable Prophylaxis Angle, or a conventional metal angle with a disposable prophylaxis cup, fill the cup by pressing the face of the cup into the paste while running the angle at a low speed. Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating the tooth surface.

Rinse tooth clean with water. Evacuation is recommended to avoid excessive dilution and swallowing of the paste.

COMPOSITION

Medicinal ingredients: None

Non-medicinal ingredients: Glycerin, Pumice, Monosodium Phosphate, Hydroxyethyl Cellulose, Sodium Saccharin, Titanium Dioxide

PROPHYLAXIS HOLDERS CLEANING AND STERILIZATION:

The Prophylaxis Holders can be cleaned and then steam sterilized, see instructions below. Discoloration may occur after repeated sterilization cycles – discoloration does not affect function.

Cleaning:

- Remove and discard the single-use prophylaxis cup.
- In an ultrasonic cleaner, prepare an instrument cleaning solution per the manufacturer's recommendation.
- Immerse the holder in the ultrasonic cleaner containing the prepared solution for the contact time specified by the solution manufacturer.
- Operate the ultrasonic cleaner in accordance with the manufacturer's instructions.

- Rinse the holder under running tap water for a minimum of 30 seconds and allow to dry completely.
- Visually inspect the holder for visible soil and repeat the cleaning procedure if necessary.
- Dry with a clean cloth or allow the holder to air dry. Proceed to Sterilization.

Sterilization:

- Place the holder in a steam sterilization pouch. Sterilize metal and plastic components in separate pouches to avoid damage.
- Place bagged holder into the steam sterilizer per manufacturer's instructions.
- Gravity Steam Sterilization: Cycle at 132°C (270°F) for 10 minutes
- Pre-vacuum Steam Sterilization: Cycle at 132°C (270°F) for 3 minutes
- To dry, use the drying cycle of the sterilizer. Set cycle for 20 – 30 minutes.
- To maintain sterility, holders should remain in the sealed pouch until ready for use and stored in a dry, clean location.

STORAGE

Store at normal room temperature 20°–25°C (68°–77°F).

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